


App Restart



If you are experiencing a problem with the Ortesi Foot & Ankle App, or the 3D Sensor is not being detected by the Ortesi Foot & Ankle App, and simply clicking the Home button doesn't do the job... try an App Restart.

How to force close an App

1. Open the multitasking and control screen by double-clicking the home button at the bottom of the iPad. This is the round physical button just below the iPad's display and screen will appear with a reduced view of the Ortesi Foot & Ankle App.
2. Hold your finger on the reduced view of the Ortesi Foot & Ankle App and then slide your finger to the top of the iPad without ever lifting your finger from the iPad's display. This gesture will force-quit the Ortesi Foot & Ankle App.

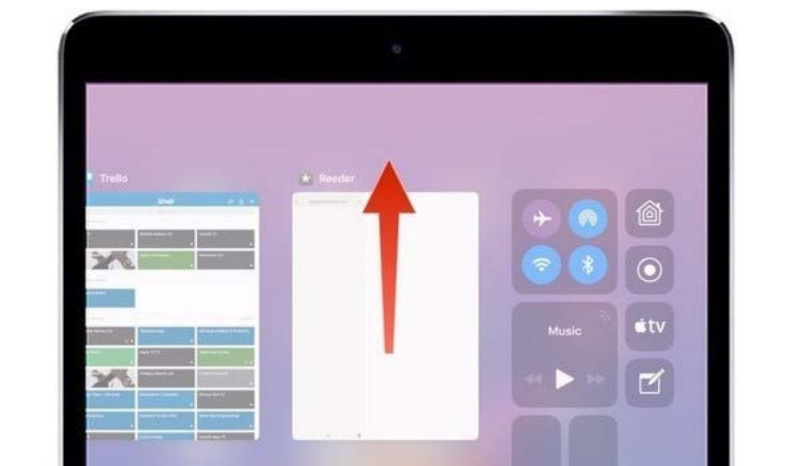

Important

To quit the app, you must drag the reduced view of the Ortesi Foot & Ankle App, not the app's icon that sits in the bottom dock of the home screen.

Need more help? Contact us on 1300 535 645 or contactus@ortesi.com.au

ortesi

3D printed braces & orthotics

Ortesi Pty Ltd
ABN 14 153 078 936
P.O Box 4555
Dandenong South VIC 3164
P 1300 535 645
F 1300 515 685
contactus@ortesi.com.au